CE

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Congratulations on your purchase of the Virmedic harness crane. The harnesses are made up of a wide range of products of different models and sizes, designed to cover the needs of any user, in accordance with their disability and on the type of transfer to be undertaken.

Our products comply with the requirements established in standard UNE-EN ISO 10535:2006: "Cranes for moving people with disabilities. Test methods and requirements".

Product in compliance with the European Guideline on Sanitary Products 93/42/CEE

#### WARNINGS AND RECOMMENDATIONS

- The harness chosen must be of an appropriate size and suited to the user's disability.
- Do not use the harness if damage to the fabric, seams or materials are observed.
- Do not expose the harness to any heat source or allow it to come into contact with flames.
- Make sure that the straps are positioned correctly in the hanger ring.

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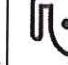
Virmedic slings have been designed and manufactured for use with Virmedic hangers

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- The harness is made of polyester mesh fabric covered with PVC or quilted polyester fabric and its straps of polypropylene roll. When disposing of it, recycling regulations or local recommendations should be followed.

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**(ENG)** This VIRMEDIC product is guaranteed for 1 year from the date of purchase under normal conditions of use, provided that the instructions are duly followed and that none of its components are changed or tampered with. If a defect in the product is detected, the distributor from whom it was purchased should be notified presenting him with the invoice or receipt, so that he may inspect it and, where appropriate and always with the manufacturer's agreement, decide on whether to repair or replace it providing that it is still within the guarantee period.

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